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EN Liquid for root canals rinsing
PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE
TO BE USED BY DENTIST ONLY

COMPOSITION: sodium hypochlorite of sodium (2% of active chlorine), water Ph. Eur.

INTENDED USE

During the mechanical canal widening it removes the remains of dead pulp. It cleans the canal and removes the smear layer to expose the dentin root canal orifices before final filling.

Adaptator used for taking the liquid preparations into the syringe

1. Unscrew the bottle cap.

2. Screw the adaptator on the bottle.

3. In order to use the bottle, remove the blue plug, screw the end of the syringe into the adaptator and tighten the screw.

The product is opened up and the liquid is introduced into the syringe, while simultaneously holding down the adaptator. Next, put the bottle upright on the table top and gently unscrew the filled syringe from adaptator. It is recommended to use luer-lock syringes with a rubber plunger.

4. After each use, close the adaptator opening with a blue plug.

5. For transport, remove the adaptors and screw the cap on the bottle.

INSTRUCTIONS FOR USE

Insert CHLORAXID 2% into the root canal using the syringe with a needle with a side opening and start rinsing.

Sodium hypochlorite in the product inactivates immediately in contact with organic substances. Therefore, to compensate, the amount of the preparation should be increased to about 10 ml per canal.

PRECAUTIONS

During root canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues.

Special care must be taken so that the product is applied into perforations. Irritating to the mucous membranes of the mouth, eyes and skin.

In case of contact rinse immediately with plenty of water and seek medical advice. In case of consumption do not induce vomiting. Drink plenty of water. Seek medical advice. Use rubber-dam when working with the product. The use of personal protection equipment (gloves, mask, gants) is recommended.

DISPENSER NIE INTENDOWANY DLA LIQUID STORAGE.
Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE
Store in original packaging at 2-8°C (in a refrigerator). Store in a securely closed packaging. Protect from sunlight. Keep out of reach of children. Expiry date is placed on the main container.

Once the container is open, the expiry date doesn't change if it the container is securely closed after each use.

TRANSPORT

The manufacturer allows transport of the product for a period not longer than 10 days at a temperature not exceeding 40°C. Short-term transport within the given temperature range does not affect the safety and efficacy of the product throughout its shelf life.

HANDLING THE PRODUCT CONTAINER
Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING

Bottle of 200 g or 400 g of preparation, adapter with a screw cap, dispenser.

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